

ZEPF Contrast PA Probe Inserts



The new **ZEPF** Contrast PA Probes are made of plastic material and dispose of a flexible working tip which adapts optimally to the anatomy of the pocket depth when measuring.

Colorstable, black markings on the white basic material guarantee a very good contrast for reading.

The sterilizable, exchangeable tips are available in different common graduations.

They are suited to determine the parodontal status and especially to be used on implants. Scratching of implant surfaces is avoided with these probes.

The tips are reusable until they bend, the color fades or the graduation is not readable any more.



24.451.02 # CPNG 22


Periodontal Probe exchangeable graduation 2/4 /6/8/10/12 mm
M4 x 0.5 mm, PU 12 pieces.
Available figures: see rear page.

26.193.15 BIONIK Universal Handle single-ended


QUICKFIX, lightred-magenta, incl. 1 end cap
The handle is available in 10 different colors.
You find the article numbers of all different handles on the rear page.



Assembly without a tool.
Do not use any tools!

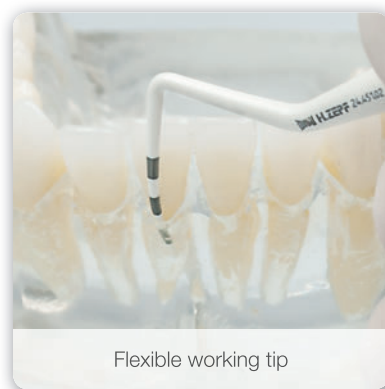

Flexible working tip

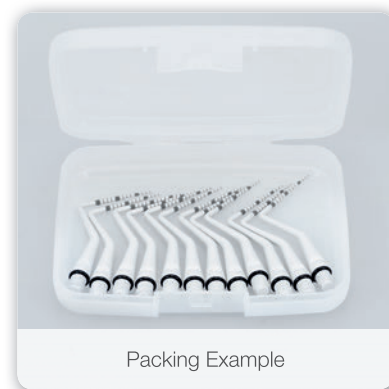

Packing Example



As our PA Probe Inserts are delicate and made of plastic, deformations can occur when sterilizing the probe under pressure or in a bent condition, or if the heat is above 275° F (135° C).

PA Probes have to be sterilized when lying in the container and without any pressure on the working tips, to avoid irreversible deformation due to heat.

For an optimum instrument preparation, we recommend the coordinated range of products from our ZEPF-Care series.








Download 

ZEPF|care

Optimum Cleaning

<http://www.zepf-dental.com/media/mediathek/protected/flyer/ZEPFCARE-E.pdf>

| Illustration | Article Description | Order Quantity |
|--|---|----------------|
|  | Contrast PA Probe Inserts made of plastic material, exchangeable, M4 x 0.5 mm | |
|  | 24.451.00 # 1 graduation 3/6 /8/11 mm | PU 12 pieces |
|  | 24.451.01 # CPG 12 graduation 3/6 /9/12 mm | PU 12 pieces |
|  | 24.451.02 # CPNG 22 graduation 2/4 /6/8/10/12 mm | PU 12 pieces |
|  | 24.451.03 # PCPG 11.5 graduation 3,5/5,5 /8,5/11.5 mm | PU 12 pieces |
|  | 24.451.06 # CNC graduation 1 - 15 in mm steps, North Carolina | PU 12 pieces |
| <p>BIONIK Universal Handle single-ended QUICKFIX for all common curette und scaler tips, incl. 1 end cap</p> | | |
|  |  26.193.11 yellow | 1 piece each |
| |  26.193.12 signal orange | |
| |  26.193.13 red-purple | |
| |  26.193.14 signal purple | |
| |  26.193.15 lightred-magenta | |
| |  26.193.16 turquoise-brightblue | |
| |  26.193.17 cobalt blue | |
| |  26.193.18 yellow green | |
| |  26.193.19 grey | |
| |  26.193.20 black | |

