3M ESPE
Cavit™
Cavit™ W
Cavit™ G
en Temporary Filling Material
de Provisorische Verschlussmasse
fr Obturation provisoire
it Prodotto per otturazioni provvisorie
es Producto para obturaciones provisorias
pt Produto para obturações provisórias
nl Tijdelijk vulmateriaal
el Υλικό για προσωρινές εμφράξεις
sv Material för provisoriska fyllningar
fi Väliaikainen täyttemateriaali
da Provisorisk fyldningsmateriale
no Provisorisk fyllingsmateriale

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en ENGLISH

Product Description
Cavit, Cavit W and Cavit G are temporary filling materials self-curing under humidity for temporary filling of cavities. The final hardness of the three variants decreases in the sequence of listing. Cavit G can be removed in one piece without residue.
Not all products are available in all countries.
🇬🇧 These Instructions for Use should be kept for the duration of product use.
Indications <ul style="list-style-type: none">Cavit for occlusal temporary fillings Cavit W with increased bonding for the temporary filling after endodontic treatments Cavit G for temporary inlays
Precautionary Measures <p>3M MSDSs can be obtained from www.mmm.com or contact your local subsidiary.</p>
Preparation <ul style="list-style-type: none">In order to prevent tubes from leaking, use the tube key to press out the paste.

- Application**
- Use an instrument to fill the required quantity into the wet cavity. The hardening process starts after a few minutes. Avoid any exposure to chewing pressure for about 2 hours after application.
 - Deep cavities need not be filled completely. If the material is used to close up a drug, pressure-free application is recommended.
 - Use rotary instruments to remove the temporary filling. Remove the Cavit G filling with a stiff probe.

- Note**
- Due to sensitivity to moisture, close the tube or jar immediately after removing the material, since otherwise the viscosity of the material increases (due to the entry of moisture), and the material may no longer be extractable.

Storage and Shelf-life
Store the product at 15–25° C/59–77°F.
Store the product in a dry place.
Do not use after the expiration date.
Use open tubes and/or screw-cap jars as soon as possible.

Customer Information
No person is authorized to provide any information which deviates from the information provided in this instruction sheet.

Warranty
3M Deutschland GmbH warrants this product will be free from defects in material and manufacture. 3M Deutschland GmbH MAKES NO OTHER WARRANTIES INCLUDING ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusive remedy and 3M Deutschland GmbH’s sole obligation shall be repair or replacement of the 3M Deutschland GmbH product.

Limitation of Liability
Except where prohibited by law, 3M Deutschland GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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