

One Step Strep A Swab Test

Wondfo One Step Strep A Swab Test is a self-testing adopting rapid chromatographic immunoassay and as qualitative screening in vitro diagnostic test with detection of group A streptococcal antigen from throat swab specimens.







SPECIMEN



STORAGE TEMPERATURE



Throat swab

4°C~30°C



TIME TO RESULTS



SHELF-LIFE

10 mins

24 months

Guangzhou Wondfo Biotech Co., Ltd.

No.8 Lizhishan Road, Science City, Luogang District,

510663, Guangzhou, P.R.China E-mail: sales@wondfo.com.cn Tel: (+86) 400-830-8768 Website: en.wondfo.com













TEST PROCEDURE

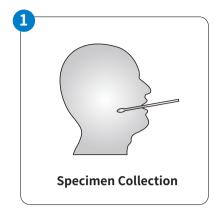


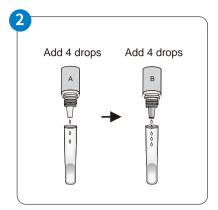

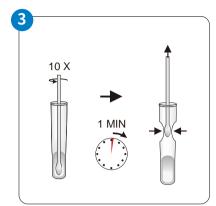

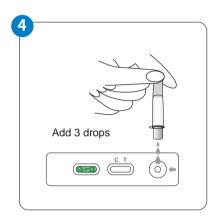



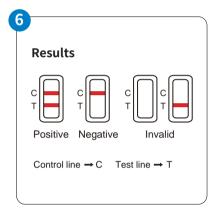

PERFORMANCE CHARACTERISTICS (SELF-TESTING)

Wondfo One Step Strep A Swab Test	Commercial Kit		Total
	Positive	Negative	iotat
Positive samples	112	10	122
Negative samples	4	923	927
Total	116	933	1049

Total agreement:

(112+923)/(116+933) ×100%=98.7%

Sensitivity:

112/116×100%=96.6%

Specificity:

923/933×100%=98.9%

ORDER INFORMATION

Product Name	Item Code	Intended Use	Package Size	Qualification
One Step Strep A Swab Test	W039P0001	Self-testing	1 Test/kit	(€ ₀₁₂₃
	W39-CH	Self-testing	2 Tests/kit	C € ₀₁₂₃